


Optimizing the blood analysis process:



THE i-STAT® DISPENSING TIP PROVIDES AN EASY CARTRIDGE-FILLING SOLUTION

THE i-STAT DISPENSING TIP:

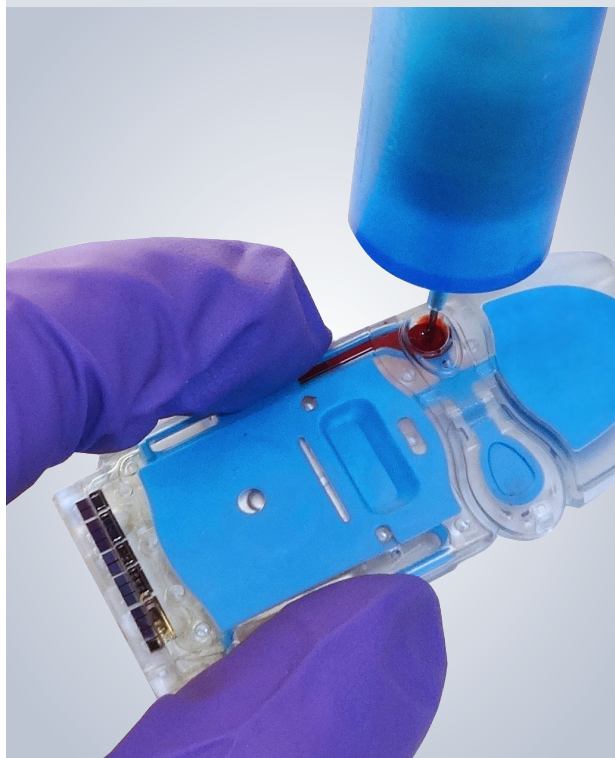

- **is fast, safe, and easy to use**
Simply slide the Dispensing Tip onto any type of evacuated phlebotomy tube to dispense blood into the *i-STAT* cartridge; the Dispensing Tip allows the tube to remain closed, reducing the risk of blood-aerosols
- **allows accurate, controlled blood application**
The end of the Dispensing Tip aligns with the cartridge port to allow the end-user to accurately dispense the desired amount of blood
- **eliminates unnecessary steps**
Intuitive design streamlines the blood-transfer process

Designed with the *i-STAT* end-user in mind, the *i-STAT* Dispensing Tip streamlines the blood transfer process to the *i-STAT* cartridge

PORTABLE BLOOD ANALYSER

i-STAT® System

BE THERE. BE CONFIDENT.



FOR IN VITRO DIAGNOSTIC USE ONLY

Not all products are available in all regions | This material is only for use outside the US

SIMPLE STEPS: OVERVIEW USING THE i-STAT DISPENSING TIP



See *Instructions for Use* for more detailed directions and follow *Universal Precautions* when handling blood.

For *in vitro* diagnostic use only.

Not all products are available in all regions; check with your local representative for availability in specific markets.

ORDERING THE I-STAT DISPENSING TIP IS EASY

To order, call 1-800-323-9100 or contact your Abbott Point of Care sales representative

PART NUMBER	DESCRIPTION	PACKAGING
06F24-20	i-STAT Dispensing Tip	100/box